



DIGITAL WORKFLOW

CONNECT TO US

- To connect with Qualident Dental Lab you will need to contact 3M Customer Service at 1.800.634.2249. Select option 1.
- You will be asked to provide contact information and an email address for your office. A temporary password will be emailed to this address.
- You will be sent a link at the address you provided. Click this link to create your 3M Connection Center account.
- After creating your account, you will be asked to choose a laboratory for your first Trusted Connection.
- Request Qualident Dental Lab under zip code 98661.
- Qualident will now appear in your settings as a Trusted Connection.

SCAN AND SEND YOUR CASE

- Turn on your scanner and open your 3M True Def Software. Select your user profile.
- Create a new patient profile and fill out the required RX information.
- Scan the preparation.
- Press the green check mark to accept the scans.
- Click the 'Prescription' button.
- Select your desired return date. When prompted to select a lab, choose 'Qualident Dental Lab' from your list of trusted connections.
- Enter any additional notes, if necessary. Next, select your desired restoration and material, then press 'sign', followed by 'ok'.
- Visit <http://casemanager.3m.com/en/login> to check the status of your cases.