



DIGITAL WORKFLOW

CONNECT TO US



- To connect with Qualident Dental Lab you will need to contact the iTero customer support team at 800.577.8767 or email iterosupport@aligntech.com.
- Tell their representative that you wish to add 'Qualident Dental Lab' to your list of favorite labs.
- Qualident Dental Lab will now appear in your settings as a manufacturing site.

SCAN AND SEND YOUR CASE

- Under Treatment Information, select Qualident Dental Lab in the ship to field.
- Scan the preparation.
- Press the 'send' button to send the case to Qualident Dental Lab.
- Your case will now be listed in the iTero Case Manager.
- We will be notified that your case is ready and we will start designing.
- Case Manager will list the current status of your case.